



COMODITÀ

Excellence in design and safety



COMODITÀ Piuma *all terrain*

STYLISH ULTRA LIGHT ALL TERRAIN ROLLING WALKER
WITH SEAT AND LARGE WHEELS

model number: COM 300

About your COMODITÀ Piuma *all terrain* Rolling Walker

COMODITÀ is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

The modern patented design and sturdy construction of the COMODITÀ Piuma *all terrain* Rolling Walker make it one of the lightest and most stylish walkers in the market.

Built to support weights of up to 300 lb. It is tested and approved under international testing standards for walkers and rollators.

In addition, the COMODITÀ Piuma *all terrain* Rolling Walker comes with the exclusive COMODITÀ Pisa removable cup holder, cane holder and washable nylon shopping bag, as well a comfortable seat and backrest.

The COMODITÀ Piuma *all terrain* Rolling Walker has 4 large wheels with soft PU tires, making it very easy to use indoors and also in all terrain outdoors. It can also be easily folded for storage or transportation.

Please do not hesitate to contact us through our website

<http://www.comoditausea.com/> if you have any doubts, questions or problems. We are here to help!

Other Features:

- Large 8" wheels with soft PU tires for comfortable all terrain rides.
- Wide, comfortable nylon seat.
- Easily adjustable handle height.
- Curb Climber included.
- Hand-brake locks for safety while seated.
- Cane holder included.
- Exclusive removable COMODITÀ Pisa cup holder included.
- Easily folds for storage and transport.
- Extra wide removable ergonomic backrest.
- Convenient shopping bag with light reflecting strap for added security.

Product Dimensions and Specifications:

- Unit Weight: 15 lb (7.1 kg)
- Weight Capacity: 300 lb (136 kg)
- Recommended User Height: 5'4" to 6'2" (1,62 m to 1,90 m)
- Overall Height: 32.0" to 35.7" (80.5 cm / 91.0 cm)
- Overall Length: 25.75" (65.5 cm)
- Overall Width: 24.7" (62.8 cm)
- Seat Depth: 8.9" (23 cm)
- Seat Width: 18.3" (46 cm)
- Folded Measurements: 26.4" x 10.2" x 32.7" (67*26*83 cm)
- Front Wheel Diameter: 8" (20.32 cm)
- Rear Wheel Diameter – 8" (20.32 cm)
- Seat height from floor: 21" (52 cm)
- Size of shopping bag: 15" W x 8" D x 10.6" H (38*20*27cm)

Assembly Instructions:

Assembly your COMODITÀ Piuma *all terrain* Rolling Walker is extremely simple.

1. Carefully remove the walker from the shipping carton.
2. 1. Place the rollator on a flat surface.



3. Release the plastic locking device that holds the walker



4. Open the walker and lock it in the opened position by pushing down the both sides of the seat onto the main frame until you hear a click.



5. Adjust the height of the handle to the desired position by standing upright behind the walker with your shoulders relaxed and arms hanging at your side. The handgrips should be at a height even with your wrists. Close the handle locking device until you hear a "click".



6. You will find a nylon strap on each side of the shopping bag that becomes a loop (fig 1). Locate the plastic supports on each side of the seat (fig 2). Move the nylon strap loop the plastic support (fig 3).



Fig 1



Fig 2



Fig 3

7. Insert the cup holder into its plastic support on the right side of the walker.



It is that simple. No tools required!

Operating Instructions:

To walk

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, pull the brake levers up.

To lock and unlock brakes

- To lock the brakes, push down the brake lever until the brakes "click" into the locked position, as shown below. Do not push it any further than this point.
- To unlock the brakes and resume walking pull up the brake levers until you hear a "click".



To use as a seat

- Make sure your walker is on a flat surface, not on an incline.
- **Lock** the brakes before sitting.
- Make sure your feet rest flat on the ground.

Care & Maintenance

Periodic brake adjustment

To adjust the sensitivity of the brake:

- Turn locking nut clockwise to increase brake sensitivity (reduce the gap between the brakepad and the tire).
- Turn locking nut counter-clockwise to reduce sensitivity (increase the gap between the brakepad and the tire).



Warning!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- ***Do not use the walker if brakes are not working properly.***

Safety Instructions:

Warning! Failure to follow safety instructions may result in serious injury.

- **DO NOT** use this product as a wheelchair.
- **DO NOT** transport someone while sitting on this walker.
- **DO NOT** exceed maximum weight capacity.
- **BRAKES MUST** be in locked position before and sitting on the seat.
- **DO NOT** self-propel the walker while seated.
- **DO NOT** sit on the walker on an incline.
- **INSPECT BRAKES** before each use.
- **ALL WHEELS MUST** be in contact with floor or ground at all times.
- **BACKREST IS NOT** intended to support your full weight.
- **DO NOT** use in escalators, stairs, moving sidewalks or uneven surfaces.
- **BOTH FEET** should rest flat on the ground when sitting.
- **RECOMMEND USER'S HEIGHT** from 5'4" to 6'2".
- Your COMODITA Piuma rolling walker should be revised periodically to ensure that the brakes are working perfectly and that all nuts and bolts are tight.

Warranty:

Your COMODITA Piuma *all terrain* Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase from the original owner. This warranty does not cover device failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges. Please contact our customer service at **info@comoditausea.com**.

We will provide a return authorization number. You **will** be required to include a copy of your original receipt.



COMODITA

Excellence in design and safety

COMODITA Piuma all terrain

Rolling walker with orthopedic seat model

COM 300

**Manufactured in China for:
COMODITA LLC USA**

www.comoditausea.com
info@comoditausea.com